

Vitrex[®] Compact Eject

Lancing device with push button ejection.
Suitable for use with Vitrex[®] Soft Lancets

Technical Data Sheet



PRODUCT:

Vitrex Compact Eject is a lancing device with a push button ejection. The device is clear adjustable with clear fixed end caps designed for Alternate Site Testing (AST).

VERSATILE:

Suitable for use with Vitrex Soft Lancets and other most commonly used lancet brands.

VARIABLE PENETRATION:


Six depth settings from 1 to 6.

CONVENIENT:

Simple and quick activation, reactivation and non-touch lancet removal.

The dimensions:

- Length: 103 ± 1.5 mm
- Width: 16 ± 0.55 mm

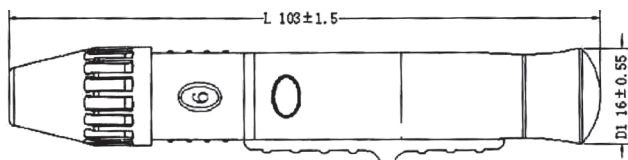

Six penetration depth presets:

Depth of penetration on place "1"	0.60mm ± 0.30mm
Depth of penetration on place "2"	0.90mm ± 0.30mm
Depth of penetration on place "3"	1.20mm ± 0.30mm
Depth of penetration on place "4"	1.50mm ± 0.30mm
Depth of penetration on place "5"	1.80mm ± 0.30mm
Depth of penetration on place "6"	2.10mm ± 0.30mm

Technical Specifications:

Product	Description	Packaging	Article No.
Vitrex Compact Eject	6-step adjustable depth	Clear plastic sleeve only	369400

MATERIALS:

The main body of Vitrex Compact Eject is made of engineering plastic ABS (Acrylonitrile Butadiene Styrene), the critical actuation part is made of POM (polyformaldehyde) and the spring is made of carbon steel and stainless steel.

PACKAGING:

Each lancing device is packed in a heat sealing bag with an instruction manual. 50 small bags are packed in one medium bag and 500 pieces of lancing devices are packed in one box.



- 1. Variable penetration**
Dial the appropriate depth on the device between 1 (min.) and 6 (max.).
- 2. Simple and quick activation and reactivation**
Pull back the grey slider to activate the mechanism.
- 3. Instant operation**
Depress the grey button to operate the device.
- 4. Lancet removal**
Simple and reliable lancet ejection.

