

MediSafe SOLO

Safety Lancets

Product features

Comfort MAX

CE 0344

● **Simple to use.**

Concave top button for comfortable thumb activation. Wide & long safety cap for easy removal. V-shape design for convenient finger position. 2-step operation. Intuitive use. No lancet reloading required. Minimum training for new users.

● **Efficient.**

Precise design and geometry of the needle provides ideal blood sample collection.

● **Sterile.** Needles are sterilized with Gamma irradiation and protected with a safety cap.

● **Safe.** Single use. Additional self-activation prevention system. Multiple sticks eliminated.

● **Patient-friendly.** Ultra sharp, thin needle. Electropolished and silicone-coated. High speed of penetration. Low puncture force.



BOX 200ct



MediSafe Solo

29G
Needle Tip

Penetration Depth 1.5 mm
Cat. No. 7130
Shelf Box 200ct



MediSafe Solo

23G
Needle

Penetration Depth 2.0 mm
Cat. No. 7140
Shelf Box 200ct



**Comfort
MAX**


Indications for use:
Indicated for blood collection for
glucose monitoring.

Sterile, single-use, safety lancets
Sterile Einmalstechhilfen
Lancetas estériles desechables
Lancetas de segurança estéreis de uso único
Lancettes jetables, stériles, garantissant la sécurité
Одноразовый безопасный ланцет, стерильный
أداة وخز معقمة ذات الاستخدام الواحد

Indications for use:
For blood collection for tests such as
glucose, hemoglobin, cholesterol, blood
type, coagulation, blood gas and many
others.

Instructions for use


1. Twist

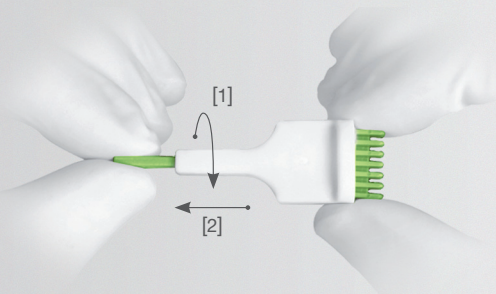

2. Press



3. Collect

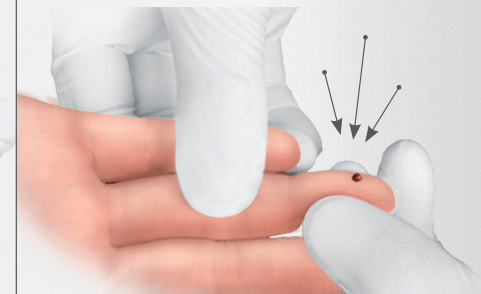

- **Twist off the protective cap and pull it straight out.**
- Die Sicherheitskappe abdrehen und gerade herausziehen.
- Gire la tapa protectora de la lanceta y retírela.
- Rode a cápsula de protecção e de seguida retire-a.
- Dévissez le capuchon et retirez le parallèlement à l'axe du piqueur.
- Снять защитный колпачок.
- قم ببلوي الغطاء الواقي ثم ابلحيه للامام.

- **Press the MediSafe Solo body firmly against the puncture site and press the push-button to activate the device.**
- Die MediSafe Solo Lanzette fest gegen die entsprechende Einstichstelle ansetzen und den Auslöseknopf betätigen um sie zu aktivieren.
- Coloque la lanceta MediSafe Solo en la zona de prueba y presione el botón a fin de activar la lanceta.
- Posicione a lanceta MediSafe Solo de maneira firme no local da picada e pressione o botão para activar o dispositivo.
- Appuyez le MediSafe Solo contre la zone de ponction et pressez un bouton pour actionner le dispositif.
- Приложите прибор к месту укола и резко нажмите большим пальцем на кнопку.
- قم بعملية الوخز بالضغط على زر الغطاء.

- **Gently apply intermittent pressure near the puncture site to obtain the required blood volume.**
- Um die gewünschte Blutmenge zu erhalten, mit kurzen Unterbrechungen sanft neben die Einstichstelle drücken.
- Apriete de maneira suave y discontinua la zona circundante al área de inyección para obtener el volumen adecuado de sangre.
- Aplique cuidadosamente, uma pressão intermitente junto da zona de picada para obter o volume de sangue necessário.
- Effectuez une pression intermittente près de l'endroit piqué pour obtenir le volume de sang nécessaire.
- Мягко нажимая на место укола, возьмите необходимое количество крови.
- اضغط برفق بجانب مكان الوخز للحصول على كمية الدم المطلوبة.

