



Groshong[®] NXT

The GROSHONG[®] PICC

Superior Performance Coupled
with Proven Effectiveness

BAIRD

The GROSHONG® PICC...



Superior Performance Coupled with Proven Effectiveness

All catheters include GROSHONG® PICC features

- **Valve Design** - reduces blood reflux and clotting
- **Three-Way Valve** - controls the flow of fluids for superior infusion therapy
- **Reduces Maintenance Costs** - weekly maintenance is reduced to a single saline flush
- **Allows for Clamp-Free Infusion Therapy** - eliminates confusion of clamping sequences
- **Easily gripped wing luer-lock connectors**
- **STATLOCK® Catheter Stabilisation Device compatible flexible wings**
- **Highly radioopaque catheter**
- **Depth markings**

Single-Lumen GROSHONG® PICC

- **Proximally Trimmable** - customises to individual patient's anatomy

Dual Lumen GROSHONG® NXT PICC

- **Enhanced Durability** - increases catheter strength
- **Equal-Sized Lumens** - enhances flow rates allowing for any type of fluid therapy in both lumens, from blood to antibiotic infusion
- **Reverse Taper Design** - strengthens the external portion of the catheter and tamponades bleeding at the insertion site

Microintroducer Kit Contents

- Microintroducer
- Guidewire 0.018 in (0.4572mm)
- Safety Scalpel
- IV Catheter, (20 Ga.)
- Safety Seldinger Introducer Needle (21 Ga.)

BLUE COLOUR
Symbolises **Saline**
Care and Maintenance

Three-Way GROSHONG® Valve



Negative Pressure:
opens valve inward, permitting blood aspiration



Positive Pressure:
opens valve outward, allowing infusion



Neutral Pressure:
valve remains closed, reducing risk of air embolism, blood reflux, and clotting



Ordering Information & Product Specifications

Type of Catheter	Product Codes	Tray	Gravity Flow (ml/hr)	Priming Volume (ml)	Gauge	ID (in)	ID (mm)
3 Fr. SL GROSHONG® PICC	7755305 7715305	Basic w/ MI Basic	246	0.22	20	0.024	0.6
4 Fr. SL GROSHONG® NXT CLEARVUE® PICC	7655405 7617405	Basic w/ MI Basic	540	0.45	18	0.033	0.84
5 Fr. DL GROSHONG® NXT PICC, 45 cm	7857505 7827505	Basic w/ MI Basic	266/266	0.35/0.35	19/19	0.026/0.026	0.66/0.66
5 Fr. DL GROSHONG® NXT PICC, 55 cm	7957505 7927505	Basic w/ MI Basic	193/193	0.41/0.41	19/19	0.026/0.026	0.66/0.66



Bard Norden AB

Box 22210
250 24 Helsingborg, Sweden
Phone: 042 - 38 60 00

Bard Norway AS

Postboks 213
1309 Rud, Norway
Phone: 815 00 266

Bard Denmark AB

Sudergade 35 B
3000 Helsingør, Denmark
Phone: 49 26 20 08

Bard Finland Oy

PL 141
00701 Helsinki, Finland
Phone: 09 347 3053

E-mail: support.nordic@crbard.com

www.bardnordic.com

Please consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions, and instructions for use.

Bard, ClearVue, Groshong, Sherlock and StatLock are trademarks and/or registered trademarks of C. R. Bard, Inc. or an affiliate.

© Copyright 2009 C. R. Bard, Inc. All rights reserved. 0509/2302